Foot massager

FootMass 5.0

medica+

JAPAN TECHNOLOGY

Instruction Manual

Foot massager
FootMass 5.0

Content

Safety instruction	01
Warranty Claims	07
Product Parts	08
Product Features	09
Product operation guide	12
Product maintenance	14
Troubleshooting	16
Product Specifications	17

Safety instruction

IMPORTANT SAFETY INSTRUCTIONS

- To ensure a safe and proper operation, please read the instruction manual carefully before using this product.
- Use this product only as described in this manual.
- Always unplug this product from the electrical outlet immediately after use and before cleaning.
- Do not use this product if you have sensitive skin, neuropathy or poor circulation. Don't let children or people with mental disabilities use the heating function without proper supervision. Improper use may be dangerous.
- Do not operate this product under a blanket or when covered. Over-heating can occur and cause fire, electric shock or injury.
- Do not cover this product in order to avoid electrical faults or damages due to excessively high motor temperature.
- Stop the massage immediately if you feel any discomfort.
- Never operate this product if it has been dropped, has a damaged cord or plug; experiences a malfunction, or if it has been

dropped in water. Please contact an authorized service centre to do the necessary check and repair.

- Do not use the components that are not recommended by Medica+.
- Do not insert any objects into any openings in the product.
- If you want to cut off the power supply, please make sure you have turned off all the operating controls, as well as remove the power plug from the socket.
- Please do not use this product as a medical device. If you need a professional diagnosis or treatment, please consult a professional medical person.

OPERATING ENVIRONMENT PRECAUTIONS

- Do not use this massager in humid or extremely dusty environments to avoid malfunction.
- Do not use this massager at a temperature above 40 degree centigrade.
- Do not use this massager outdoors.
- Avoid placing or operating this massager in direct sunlight or near a heating appliance, stove.

USER PRECAUTIONS

- Make sure that this product is far away from any hot surfaces.
- Please consult your doctor before use for those who have the following diseases:
 - Those who have heart disease, malignant tumor, back pain, spinal scoliosis, hypertension, osteoporosis, etc.
 - Those who are pregnant or having a menstrual period.
 - Those who have a physical discomfort, fever, a swelling or inflammation, bruises, allergies, etc.
 - Those who are warned by the doctor not to do any massage to avoid injury or physical discomforts such as blood clots (thrombosis), severe aneurysm, acute venous tumor, all kinds of skin inflammation, infectious skin disease (including subcutaneous tissue inflammation), etc.
 - Warnings: Do not use the massager together with the following medical instruments.
 - 1) Those who are using a pacemaker or other implantable medical equipment.
 - 2) Those who are using an artificial heart.
 - 3) Those who are wearing an electrocardiograph for similar medical equipment.Otherwise it will cause malfunction for those medical equipment, putting the user's life in danger.

PRECAUTIONS FOR POWER SUPPLY

- Do not turn on the switch when your hands are wet.
- Do not immerse in or spray water or liquid on the control power switch.

CORD/PLUG PRECAUTIONS

- Do not connect or remove the plug with wet hands. Otherwise you will suffer an electric shock or a product malfunction.
- Do not pull the wires when you pull out the power plug. Please hold the plug itself.
- As the default operating time is 15 minutes, all the functions will stop working after 15 minutes. Please unplug this product from the electrical outlet immediately after use.
- Keep the cord away from hot surfaces.
- Do not place the power plug under the product or any heavy items.
- Do not wind the power plug around the product lest it should be damaged and cause an electric shock or fire.
- Do not operate this product if it has a damaged cord or plug.
- Do not use this product if the power socket is loose.
- Do not carry this product by its supply cord or use the cord as a handle.

■ When the power plug is damaged, please call the service centre or an authorized technical person for the necessary check and maintenance.

ATTENTION


- Do not fall asleep while massager is on.
- Do not use this product while you are drunk or physically uncomfortable.
- Don't stand on the product or put a heavy weight on it.
- Do not use this product for more than 30 minutes.
- Do not use this product if the cloth cover is worn out.
- Do not use this product within an hour before or after a meal.
- When you are using the product at a low room temperature, do not suddenly use the heat function. You can adjust the room temperature to normal room temperature, before starting the heat function.
- When moving this product from a low temperature storage place to a warmer setting, it may cause malfunction due to water vapor gathered in the components. Set the product in a normal room temperature for one hour before using it.

- Do not use this product continuously if you find any components damaged or the product in an abnormal condition.
- When current interruption occurs during your massage, try to avoid further damage due to sudden power restoration.
- Stop using the product if there is thunder and lightning.

Warranty claims

1. Warranty is not transferable.
2. This warranty is effective only if the product is purchased from Medica+.
3. Proof of purchase (original receipt) is required for all warranty repairs.
4. All implied warranties, including but not limited to these implied warranties of fitness and merchantability, are limited to the duration stated in the warranty policy.
5. Damage due to misuse, improper treatment and unauthorized modification and repair are not covered by this warranty.
6. Warranty does not cover accessories and attachments which do not belong to the massage unit.
7. Warranty is not effective to rental, business, commercial, institutional or other non-residential users.
8. All services covered by this warranty must be approved by Medica+ and repaired by authorized technicians only.
9. If replacement parts for defective materials are not available, Medica+ reserves the right to make substitution in lieu of repair or replacement.

Product Parts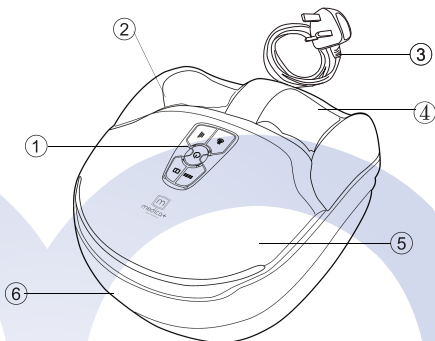


- 1. Control panel
- 2. Zipper
- 3. Plug
- 4. Detachable fabric cover
- 5. Upper cover
- 6. Lower cover

Product Features

- Specially designed for foot care; U-style wrapped air compressing massage with a comfortable warm heating function, to help relieve foot pains and discomfort.
- New programmes: Tone/Revive/Reflexology programmes are designed for different groups, to protect your foot, relieve your pressure, and ease your body and mind.

Product operation illustration

VIBRATION: Press on/off, It will control vibration function. Lights on, vibration function on, Lights off, vibration function is off. Under stand by mode, press vibration for 3 seconds, it can solely control vibration. 3 modes of vibration adjustment, Blue lights on for mode 1. Red lights on for mode 2. Green lights on for mode 3.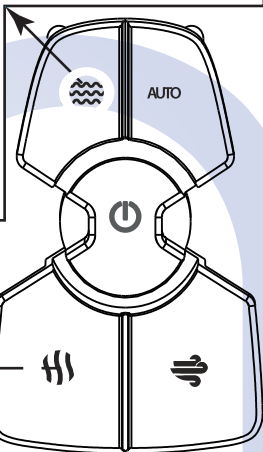


HEAT: Press the button, Red lights on for heating function. Lights off for close heating function.

Note: 1. when you turn the vibration function keys to adjust the intensity of the three levels of vibration.

Note: 2. under automatic massage function, intensity key is for adjust the intensity of air-compression.

PRODUCT OPERATION GUIDE: Auto program modes: Place your foot in the product ,before pressing the switch key on the screen. When the blue light is on , is mode ' TONE'; Red light is on, is mode 'Revive'; Green light is on , is mode 'Reflexology'; program.

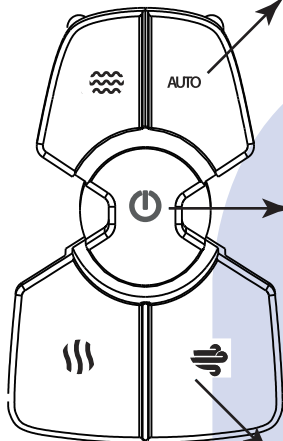ON/OFF: Control the product power switch, when in standby mode, the red light flashes; push this button, the green light on, all functions like program, vibration, heating, etc. are available to choose, push it again to turn off the function.

INTENSITY: 3 levels of air-compression intensity adjustment, green lights on for low level. Blue lights on for media level. Red lights on for strong level. Light off for close function.

Product operation Guide

a) Instructions before use

Please store this product in a dry place, do not expose the product to direct sunlight. Put the carpet under the product to minimize the working noise.

b) How to use it

This product uses only the 100-240V, 50/60Hz power supply. Make sure your indoor voltage fits this specification parameter.

- Connect the AC power cord to the socket.

*do not insert or remove the plug with a wet hand

Do not put your hands inside the cloth cover.

- Place your foot in the product, before pressing the switch key on screen. When the blue light is on, the other functions will follow. According to the indicators on the control panel, you can choose different massage modes.
- When you start the massage, the product will be countdown in 15minutes. After 15 minutes, the massage function is turned off, and it will recover to standby mode.
- When the product is on standby, press the main switch “0” to cut off the power.

Product maintenance

A) STORAGE

- Please store it in the packing carton, and put in a safe, dry, well-ventilated, and shaded place.
- Do not twist the power plug.
- Do not hang the power wire and pull the product.
- Avoid sharp items from destroying the surface of the product.
- Please don't store the machine under direct sunlight or in places of high temperature.

B) CLEAN

- Before cleaning, make sure that the power cord is unplugged from the electrical outlet and that the machine is completely cool.
- Use a damp cloth to clean the product. Ensure the product is completely dry before reuse.
- The detachable fabric cloth can be washed in a normal washing way.


*ATTENTION

Make sure not to use benzene or thinner and other corrosive detergents to clean the product.

C) MAINTENANCE

- Keep the product clean and store in a dry environment.

- When the product is not in use for a long time, please put a dustproof cover on the product.

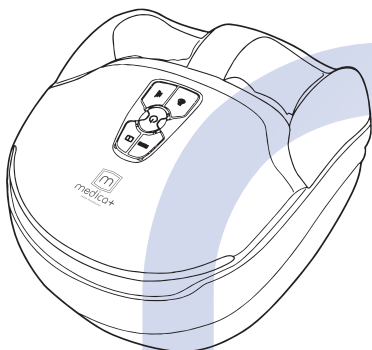Troubleshooting

If you meet the following problems, please referring to the following solutions. If the product is still malfunctioning or doesn't work, please send the product to our service centre or an authorized technician for the necessary checks and repair.

IT DOESN'T WORK

1. Plug isn't connected correctly to the socket.
Push the plug firmly into the socket.
2. The power switch at the back is off.
Switch on the power switch at the back of the product.
3. Switch key on control panel isn't on.
Switch on the on/off key on the panel.
Switch on the on/off key on the panel.
4. The fuse blows out.
Insert a tooth pick in the small hole of "fuse" under the switch power, before taking out the fuse.

STOP WORKING SUDDENLY

1. The power switch is off.
Check that the power plug is connected well.
2. Massage time is over 15 minutes.
Restart the power supply switch.

3. If it is working for a long time, the motor will be protected against overheating.

After the product is cooled down, please restart the ON/OFF power.

4. If you put heavy pressure on the rollers and the rollers do not work well.

Switch off the power supply, and turn it on again.

Product specification

Product name	Foot massager
Model number	FootMass 5.0
Voltage	100-240V, 50/60Hz
Power consumption	65W
Timer	15minutes

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