Professional Alcohol Tester

Alco Control 7.0

medica+

JAPAN TECHNOLOGY

Instruction Manual

**Professional Alcohol Tester
Alco Control 7.0**

Content

Introduction	01
Safe & High Quality design	01
Parts instruction	02
Operation descriptions	03
Breath test instruction	04
Low battery detection	10
Packing accessories	12
Common faults and solutions	13
Unit comparison table	14
Cautions	15
Important notes	16



Introduction

Thank you for purchasing Alco Control 7.0 professional breathalyzer for home use. Our FDA cleared alcohol breathalyzer adopts the latest advanced high-precision sensor technology to produce professional-grade accurate readout in seconds. Please ensure you read the manual in its entirety before operating.

Safe & High Quality Design

- The design and important parameters such as device sampling method, numerical accuracy, and display range refers to the EN 16280 standard drafted and officially voted by the EU in the field of breath alcohol testing devices used by the general public.
- The device is designed with three units of measure mg/L, ‰BAC, %BAC, which you can easily switch between.
- Designed with a single button operation for easy use.

Parts Instruction

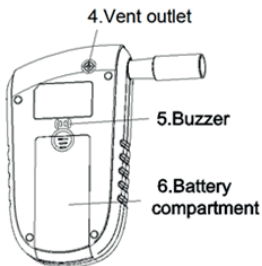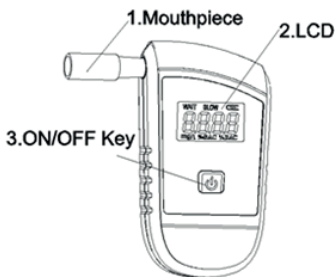1. Mouthpiece
2. LCD
3. On | Off Key
4. Vent outlet
5. Buzzer
6. Battery compartment

Operation Instruction

1. How To Use

Rinse your mouth with water before using. Avoid smoking or drinking alcohol for at least 20 minutes before using.

2. Power ON

Install 2* AAA alkaline batteries into the device's battery compartment, press and hold the key for 2.0 seconds to turn on and it will enter into the self-checking mode automatically.

3. Power OFF

In the test result reading mode, the device will turn off automatically if left idle for 30 seconds.

Breath Test Instructions

4. Intelligent self-checking

Turn on the device, it will enter into self-checking automatically to reset the sensor to ensure accurate reading, the following picture will be displayed.

5. Wait for preheating

After self-checking, device enters into the preheating mode to activate the warming of the sensor, countdown 19s (19-00), this is to stabilize the sensitivity of the sensor.

Note: If the device has not been used for an extended period of time, it is recommended to clear out the back-flow of expired air from the device by conducting the first blow, this is to ensure any trapped air is cleared out of the device before use.


6. Breath sampling

After 19s of preheating, “BLOW” flashes for 25 seconds with beep tones and the device is ready to breath test, please carry out the measurement within this period, otherwise the device will power off automatically.

When sampling, place the mouthpiece into your mouth, blow continuously for 5s and keep press On | Off key at the same time while blowing, only stop blowing once the beep tone stops together with status bar full displayed. Two short beep sounds will be heard when the sample collection is successful, test result will be displayed accordingly.

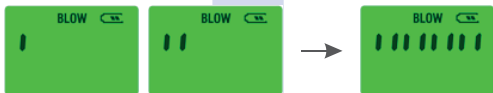- During breath sampling, to make the test result more accurate, please blow into the mouthpiece slowly and consistently.
- Once the device has completed taking a breath sample, you can stop blowing.

At the test result display interface, you can press On|Off key to re-test, or keep press On|Off key for 2.0s to switch the measurement units between mg/L, ‰BAC, % BAC. The next test result will be shown with the unit you selected.

6. Breath sampling

Alarm value with factory default setting (Generally there is no need to set up again). If you need to change the alarm value, in the self-checking and preheating mode, press On|Off key twice to enter into the alarm value setting mode, and short press On|Off key to select the alarm value as b2, b3, b5, b8, after selected the device will go back to test mode automatically after 3 seconds.

Corresponding critical value and meaning are as following:

4 configurable alarm level values.and pull the tip from the handle.

Corresponding critical value and meaning are as following:

4 configurable alarm level values.and pull the tip from the handle.

$C \geq 0.20\% \text{BAC}$
or $0.020\% \text{BAC}$
or 0.100mg/L

B2

$C \geq 0.30\% \text{BAC}$
or $0.030\% \text{BAC}$
or 0.150mg/L

B3

$C \geq 0.50\% \text{BAC}$
or $0.050\% \text{BAC}$
or 0.250mg/L

B5

$C \geq 0.80\% \text{BAC}$
or $0.080\% \text{BAC}$
or 0.400mg/L

B8

Tip: If customer has no special requirements, the factory default setting of alarm value should be used.

7. Test result show

If the test result exceeds the alert level, the backlight turns red and flashes 10 times together with a beeping sound prompt.

If the test result is lower than alarm level, the backlight will remain Green.

The reading range is 0.00‰BAC -2.00‰BAC.

“2.00‰BAC” will be shown if the reading is higher than 2.00‰ BAC.

“0.00‰BAC” will be shown if the reading is lower than 0.05‰ BAC.


Take alert level set as 0.50‰BAC as an example, test result will show as the following pictures:

8. Test records review

Device will record the 10 most recent testing results. In the intelligent self-checking and preheating mode, hold On|Off key for 2.0s to enter into record memory mode. After entering, press the On|Off key repeatedly to review the last stored records (max. 10 records), the times of record and test result will flash on the display. The device will switch back to test mode automatically if there's no key operation within 3 seconds.

For example, if the 1st record is 0.50‰ BAC, then “-01-” and the BAC reading “0.50‰BAC” flashes alternately; if the 2nd has not been tested, then the recorded data will show “----”. (As per the following pictures):

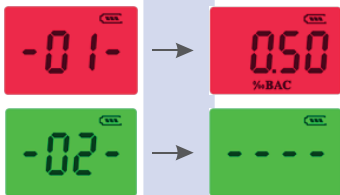Note: When the batteries are taken out, the stored records will be cleared.

Low battery detection

When there is a low battery level, “Lob” flashes with sound prompt as following picture. After “Lob” flashes 10 times the device will power off automatically, please replace with new AAA alkaline batteries.

While low battery the device will not work normally, you need to replace with new batteries, or it may not power on.

Item	Specifications
Sensor	Semiconductor sensor
Reference Standard	EN 16280
Response Time	3-5s
Blowing Time	About 5s
Detection Range	0.00~2.00‰BAC; 0.000~0.200%BAC; 0.000 ~1.000mg/L; (If the conversion factor between ‰BAC units and mg/L units is 2.0, then 0.50‰BAC = 0.050%BAC = 0.250 mg/L)
Sensitivity	0.001% BAC
Sensor Accuracy	+/-0.010 at 0.050% BAC;>0.050% BAC: +/-20%
Record	10 test records be stored
Optimal Operating Temperature	0°C~50°C
Storage Temperature	-20°C~70°C
Display	About 1.5 inch LCD(positive)
Power Supply	2 × AAA alkaline batteries
Dimensions	110*61*24 mm
Net Weight	54 g

Packing accessories

Package Includes:

1x Alco Control 7.0 Professional Breathalyzer

1x User Manual

1x Compact carry box for easy safe storage

5x Reusable Mouthpieces

Optional accessories:

2* AAA alkaline batteries, mouthpieces.

Tip: If find that the instrument is damaged or missing accessories, please contact your supplier immediately.

Common faults and solutions

Can't power on

1. Low battery – Replace new batteries.
2. Batteries are upside down – Re-install the batteries in the right way.
3. Haven't held On|Off key for 2.0s – Hold down the On|Off key for 2.0s to turn the device on.

Two test results differ greatly in a short time

1. The vent outlet maybe blocked during a test – Don't block the vent outlet while blowing. ensure one smooth consistent blow into the device.
2. After drinking alcohol, the wait time is not long enough before taking the alcohol test – To get a more accurate test result, please abstain from smoking and drinking for at least 20 minutes before testing.
3. Before the test, no water or non-alcoholic beverages was used to rinse the mouth and alcohol residue remained in the mouth – To ensure the test result is most accurate, Please

rinse out your mouth with water or non-alcoholic drink before sampling test.

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Cautions

- Different people have different tolerances for alcohol, which is affected by factors such as heredity, physique, gender, and weight. Therefore, different people can drink the same amount of alcohol, and the test results will be different.
- After drinking alcohol, there is a process of absorption and decomposition of alcohol in the blood, so the alcohol content in your blood can fluctuate within a few minutes.
- Store the device in its packing box, avoid exposure to sunlight and humidity.

Important notes

- Don't allow alcohol or any other liquid enter into the device, or it will cause permanent damage to the sensor.
- The frequency of use and BAC test concentration of device will affect the sensitivity and life of the sensor. The more frequently used and higher alcohol concentration tested the sensor sensitivity will be reduced and service life will be shortened.
- Test results are for reference only, do not use this device to be 100% sure if you can drive or not. The test results may be inadmissible in court.
- For health and safety of you and others, please don't drink before drive.

**Neither the re-seller nor manufacturer accepts any liability arising from the use of the unit.
Don't drink before you drive!**

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