ENG / UA / PL


Manual Massager

MassHand Pro 4.0

medica+
JAPAN TECHNOLOGY

ENG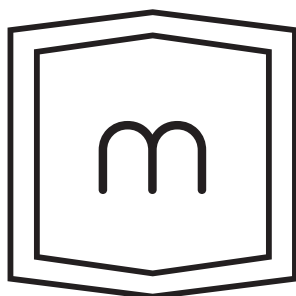

medica+
JAPAN TECHNOLOGY

Istruction

MassHand Pro 4.0

Content

Precautions.....	01
Prohibition.....	01-02
Specifications.....	03
Product introduction.....	04
Massage heads	05
Multi-function button use.....	06
About charging.....	06
Troubleshooting.....	07-08
Maintenance.....	09-10
Environmental protection	11
Warranty conditions.....	12

Thank you for choosing our booster massage gun. Before using it, please read this manual carefully to ensure proper operation and keep this manual for future reference.

- Relieving muscle soreness;
- Stimulating the circulation of blood;
- Relieving stiffness of cervical vertebra;
- Improving muscle recovery;

Precautions

To avoid harm, fault, electric shock, fire, death and other equipment damage, please read the instructions and warnings carefully.

Prohibition

1. Minors are forbidden to use this product. It is not a toy.
2. Forbid to put this product into water. If it falls into water accidentally, please send it for inspection in time.
3. Forbid to put this product near or into the fire.
4. Forbid to use it on head, neck, heart or genital.
5. Forbid to use it on bones. Use it on soft tissues only when not feeling pain or discomfort.
6. Forbid to use non-original charger to charge the product, otherwise it may cause a fire.

7. Forbid to drop the product intentionally.
8. Forbid to remove the screws or try to disassemble the product.
9. The product is not an alternative or complement of medical treatment. If you are feeling pain, undergoing medical treatment or using any other medical devices, please consult your doctor before using this product.
10. Forbid to put fingers, hair or other body parts near to the back of the massage heads in case of the squashing.

Do not touch this part when the machine is running without the massage heads being installed.


Do not touch this part when the machine is running with the massage heads installed.


Specifications

Product name:	Mass Pro 4.0
Type:	mini fascial gun
Charging interface:	Type-C, 5V 1A
Rated voltage:	7.4V
Battery capacity:	1500 mAh
Net weight:	0.433 kg
Size:	173×111×58 mm

Product introduction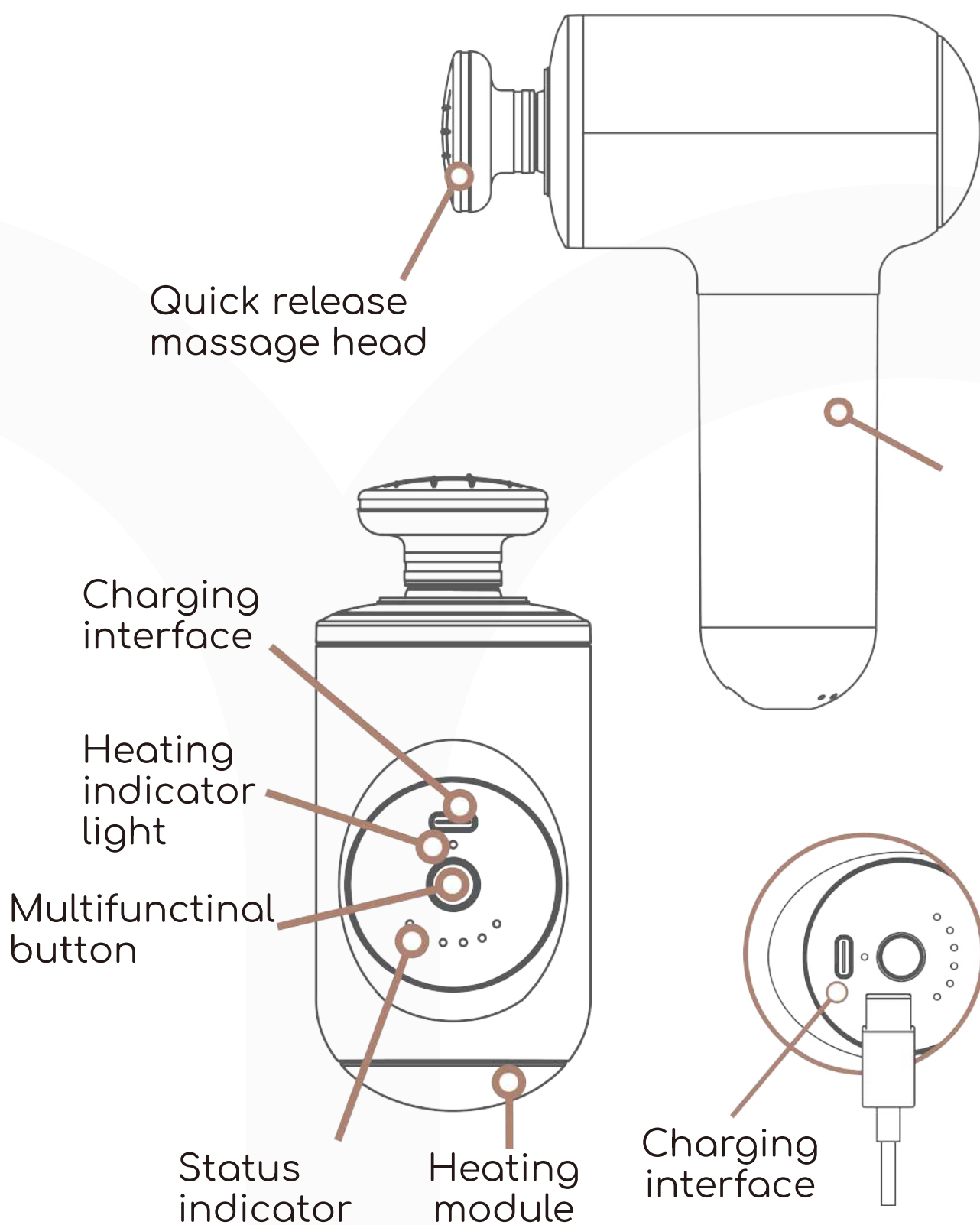

Massage heads

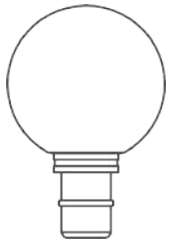
Round head

For large muscle group massage and relaxation.

Bullet head

For pain point relaxation. Strong intensity.

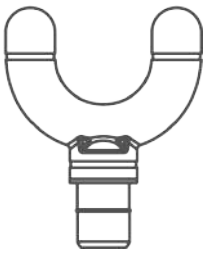Spinal head

For spine massage. Strong intensity

Flat head

Suitable for massaging all muscles

The charging port is located at the bottom of the fascia gun.

1. For the first use, please fully charge the battery of the machine.
2. Turn off the device, connect the Type-c charging cable, and the 1-6 bars will flash in sequence when charging.
3. All 6 LEDs are on when fully charged.
4. When charging, please do not start the machine.

Multi-function button use

1. Long press to power on. (The power display is 6 grids, color: white). All six lights flash when the battery is exhausted.
2. Short press to switch gears 1-6 - heating gear.
3. In the heating state, the red light of the heating status indicator flashes.
4. Long press to power off.

About charging

When assembling or replacing the massage head, please insert the massage head directly into the connection of the muzzle of the fascia gun until the alignment line is aligned with the muzzle.

When the massage head is replaced, pull it out directly or pull out the massage head by rotating it.

Troubleshooting

Can't start or won't start

1. The temperature is too low:

If the fascia gun is placed in an environment below 0 degrees Celsius for a long time, the battery activity of the fascia gun will decrease and it will not start normally, the whole machine shows that it can be turned on normally, but cannot operate normally.

Solution: Stand still for 2 hours in an environment above 0 degrees celsius, or use a hair dryer to quickly increase the body temperature.

2. Low battery power:

When the battery is in low power, if the customer does not charge for a long time, the battery will enter low power protection state, if you start the machine again at this time, the fascia gun will not work normally.

Solution: keep the power off, connect the charger, and make sure that the battery can be used normally after the battery is fully charged.

The product does not work

Check whether the battery is sufficient, and it needs to be fully charged for 8 hours if it has not been used for a long time.

The panel lights are off.

Check whether the battery is sufficient.

There is a slight abnormal noise in use

The running-in period of the new machine is a normal phenomenon. If the sound becomes louder after using for a period of time, please send it for inspection.

The battery won't charge

The battery is aging or poor contact, please replace the battery.

Maintenance

1. When not in use for a long time, keep 1-2 bars of power, turn off the power switch, keep the product properly, and put it back into the original bag

Packed in a box, placed in a safe, dry, ventilated, low temperature place.

2. When not in use for a long time, it is recommended to charge the product every six months to prevent the battery from being damaged due to power loss.

3. For daily maintenance, please use neutral detergent and soft cloth to wipe, do not use engine oil, organic solvent or other chemicals

Do not wash with chemical agents, let alone rinse in water.

4. Please turn off the power switch after using the product or before cleaning it.

5. The product should be fixed during transportation to prevent violent vibration, collision, rolling and falling.

At the end of the service life of this product, please do not throw it away together with general household waste, put this product in front of the recycling bin designated by the government.

You can also send this product back to the manufacturer's after-sales service center, and the personnel at the service center will dispose of it in an environmentally sound manner.

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This form is based on the rules of SJ/T 11364.

○: The content of harmful substances in all parts meet the limited requirements of GB/T 26572.

Environmental protection use period

The time limit of environmental pollution, serious damage to people and property, or the leakage of toxic and harmful substances, is 10 years in normal use as described in this manual.

Warranty conditions

Please keep this manual properly to use the warranty card. The warranty is provided for one year from the date of purchase (or for 2 or 3 years if an extended warranty is purchased) in case of malfunction under normal operating conditions.

The warranty does not cover the following cases:

- If the product is damaged due to improper use or lack of proper maintenance.
- If the device is damaged as a result of disassembly or modifications.
- If non-original parts that do not belong to our company have been applied and used in the device.
- Damage caused by force majeure circumstances.

According to the terms, maintenance services provided outside the warranty period are subject to an additional fee.

Parts replaced during the warranty period remain the property of MEDICA+.

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