**ENG / UA / PL**

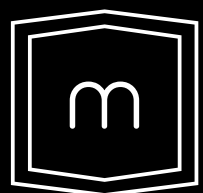
**Hearing aid**

**Sound Control 14**


**medica+**  
JAPAN TECHNOLOGY

medica+

JAPAN TECHNOLOGY

**ENG**

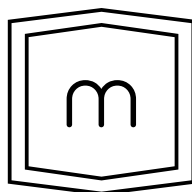**medica+**  
JAPAN TECHNOLOGY

# **Instruction manual**

## **Sound Control 14**

# **Content**

---

|                                  |           |
|----------------------------------|-----------|
| <b>Operating Principles.....</b> | <b>01</b> |
| <b>Features .....</b>            | <b>02</b> |
| <b>Installation Steps .....</b>  | <b>03</b> |
| <b>Usage.....</b>                | <b>04</b> |
| <b>Controls .....</b>            | <b>06</b> |
| <b>Charging.....</b>             | <b>05</b> |
| <b>General Precautions.....</b>  | <b>07</b> |
| <b>Warranty Terms.....</b>       | <b>08</b> |

Thank you for choosing our hearing aid! We have put a lot of effort and attention into ensuring the highest quality of the device, as well as ease of use and maintenance.

Please familiarize yourself with the information in this manual, where you will find instructions on how to install and remove the hearing aid, as well as explanations on how to operate it during daily use. Proper understanding and usage will allow you to gain the maximum benefit for your hearing.

## **Operating Principles**

The hearing aid receives signals through the microphone and then converts them into digital or electrical signals, depending on the type of hearing aid. The received signals are then amplified. When the signals are amplified, the sounds transmitted to users are also enhanced. The level of amplification varies based on different degrees of hearing loss and can be pre-set by an audiologist or adjusted using the button on the hearing aid.

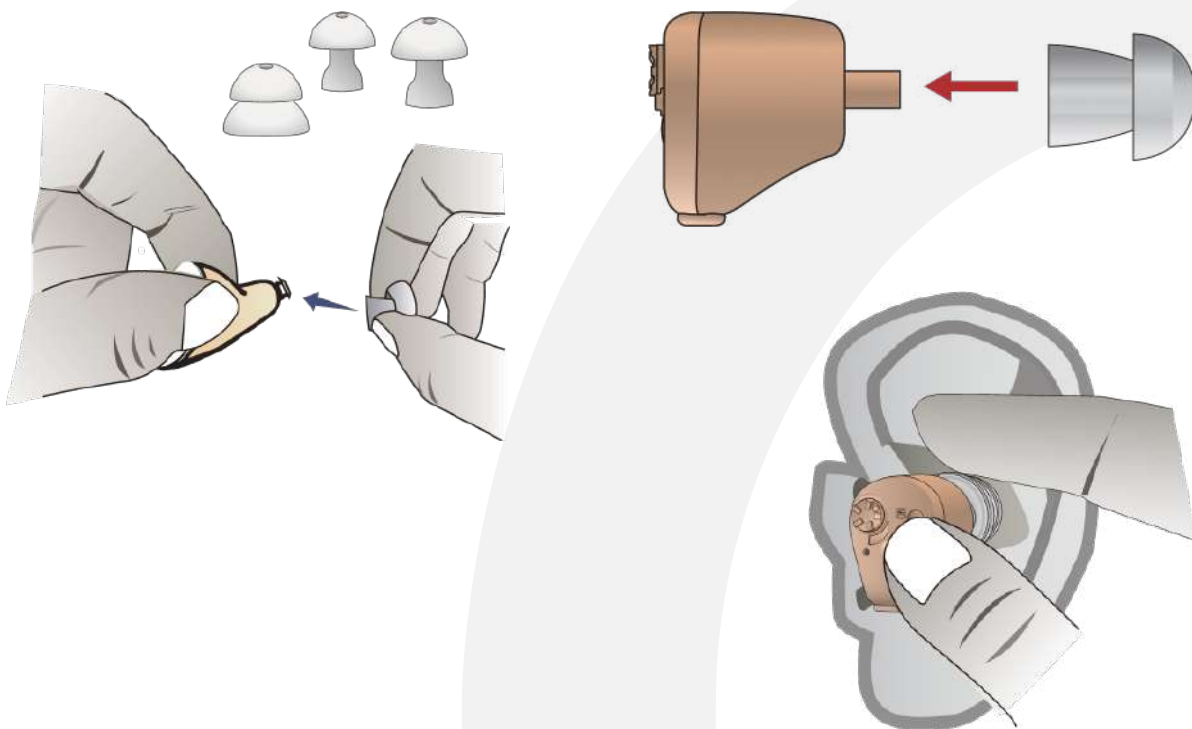
# Features

1. Microphone
2. Charging port
3. Earplugs
4. Charging indicator
5. Volume control
6. On/Off switch

# Installation Steps


1. Select the correct earplug size.
2. Connect the hearing aid to the earplugs and secure it in the inner ring of the earplug.
3. If you notice that your hearing aid is loose while wearing it, please replace the earplugs with a larger size.

# Usage

Switch the On/Off button to the "OFF" position and set the volume control to the minimum. Adjust the position of the hearing aid so that the microphone is facing your palm—this is the correct direction for wearing. Hold the device with your thumb and index finger, insert the tip of the hearing aid into the ear canal first, and slowly rotate it until it fits snugly into the ear canal.

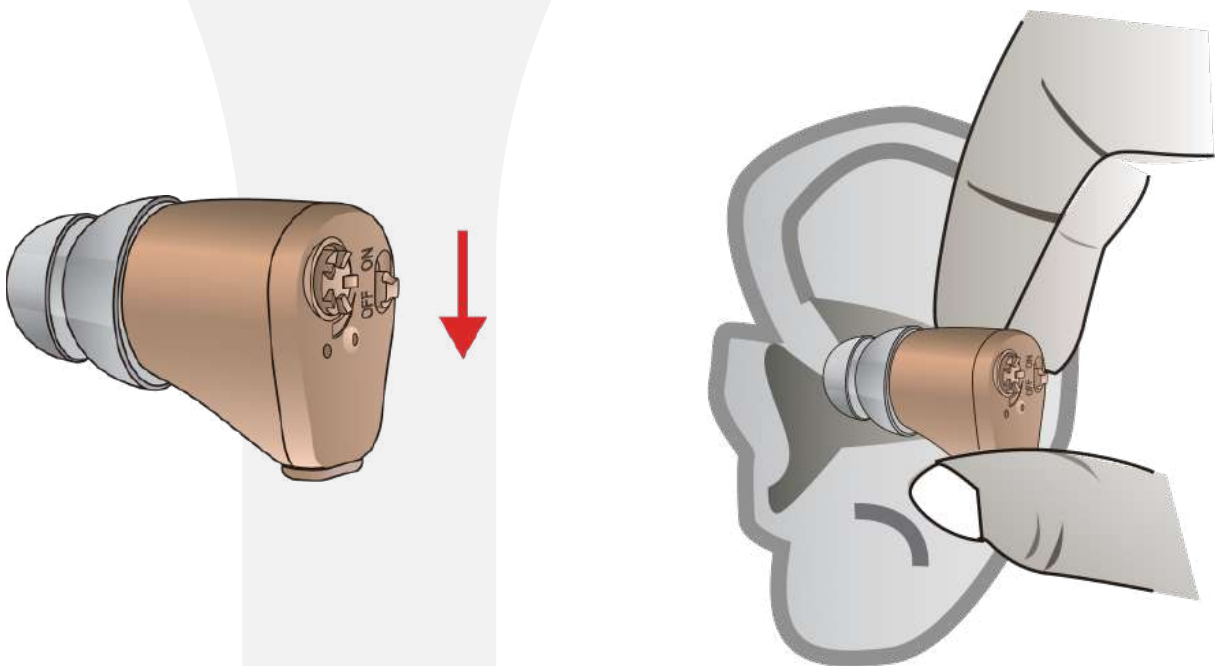
(Note: The hearing aid should be turned off, or the volume control should be at a minimum. Otherwise, you may hear a "whistling" sound when inserting the earpiece, which can cause discomfort.)

# Operation

1. Switch the On/Off button to the "ON" position.
2. Use the volume control to adjust the volume.
3. When removing the hearing aid, please reduce the volume to a minimum and turn it off.

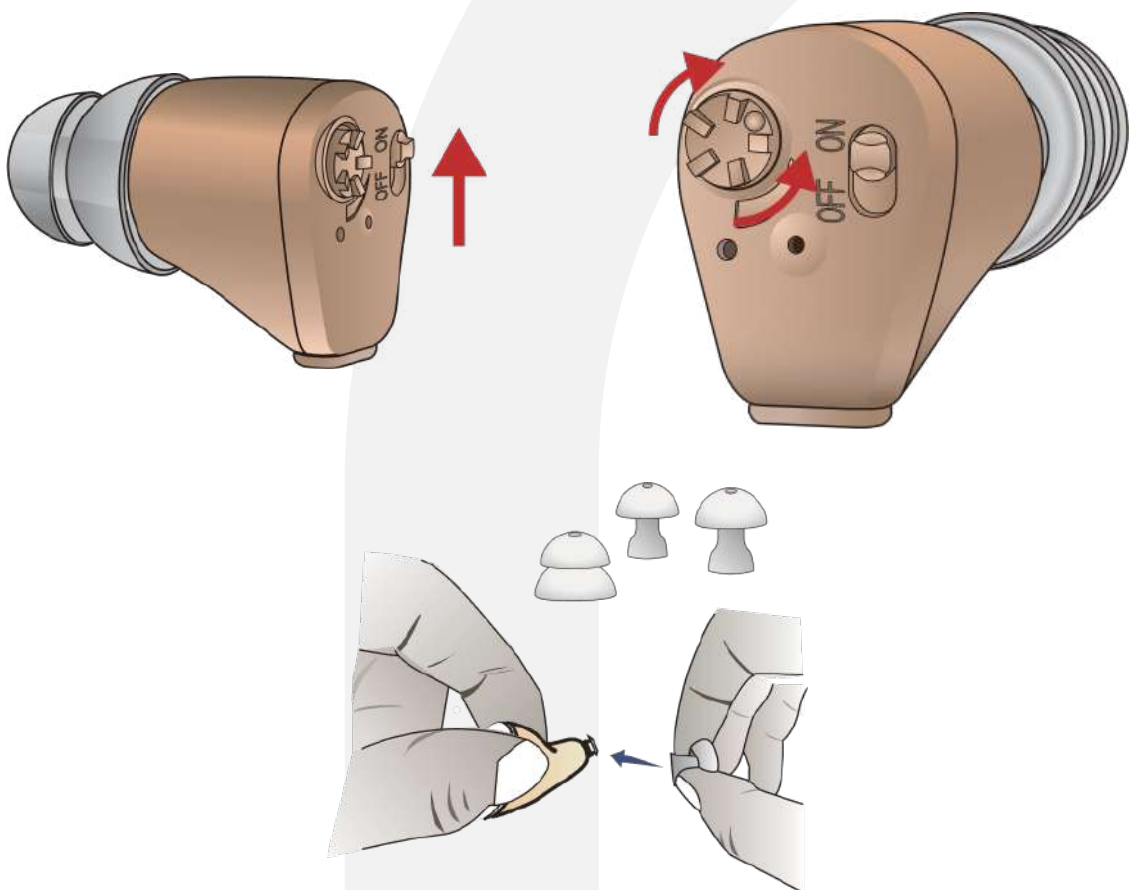(Note: If you hear a whistling sound while wearing the hearing aid, check whether your earplugs are properly secured. If they do not fit well, try replacing them with a different type.)

# Charging

1. If the volume of the hearing aid decreases or it does not function properly, the battery may be low. Please charge the hearing aid.
2. When charging the hearing aid, switch the ON/OFF button to the "OFF" position. Connect the standard end of the USB charging cable to the power adapter, plug the other end into the hearing aid's charging port, and connect the power adapter to a 220V outlet. The green indicator on the power adapter will blink, indicating that the device is charging.
3. The green indicator will stop blinking when the battery is fully charged.

# General Precautions

1. Keep the hearing aid clean and dry. After use, wipe the casing with a soft cloth or tissue to remove dirt or moisture.
2. Never immerse the hearing aid in water or other liquids, as this may cause irreversible damage.
3. Avoid rough handling and do not drop the hearing aid on hard surfaces or floors.
4. DO NOT expose the hearing aid to direct sunlight or place it near heat sources, such as inside a parked car on a hot day, as excessive heat can damage or deform the casing.
5. DO NOT wear the hearing aid while showering, swimming, in heavy rain, or in humid environments such as a sauna or steam room.

# Warranty Terms

Dear Customers!

Please keep this manual properly to use the warranty certificate. The warranty is valid for one year from the date of purchase (or two or three years if an extended warranty is purchased) in case of malfunction under normal operating conditions.

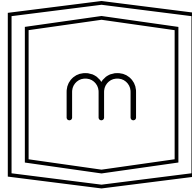
The warranty does not cover the following cases:

- Damage caused by improper use or lack of proper maintenance.
- Damage resulting from disassembly or modifications.
- Use of non-original parts or components that are not from our company.
- Damage caused by force majeure events.

According to the terms, any maintenance provided outside the warranty period is subject to additional charges.

Parts replaced during the warranty period remain the property of MEDICA+.

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