ENG / UA / PL

MassHand Pro 2X

medica+ 
JAPAN TECHNOLOGY

medica+

JAPAN TECHNOLOGY

ENG

Instruction

**MEDICA+ MassHand Pro 2X
Percussion Massager (Japan)**

Content

Safety Instructions.....	01-02
Device Description	02-04
Attachments.....	05-06
Charging	07
Instructions for Use	08
Contraindications.....	09
Heat/Cold Attachment.....	10
Warranty Terms	11
Technical specifications	12

Dear Customer!

Thank you for choosing MEDICA+ products.

Please carefully read this manual before using the device.

Safety Instructions

1. This device is not a medical product and is not intended for diagnosing, treating, preventing, or curing any diseases.
2. Before use, consult a physician if you experience pain or if you have any of the following conditions: pregnancy, diabetes, retinal diseases, neuropathy, implanted medical devices, or any other chronic or acute illnesses or medical limitations.
3. The device must be charged using a power adapter with output parameters of 5V/2A or a compatible charger of appropriate power. Fully charge the device before first use. Do not turn on the device while charging.
4. Attachments may only be replaced after the device has completely stopped operating. Do not change attachments while the device is in operation.
5. Do not immerse the device in water or other liquids.
6. Do not use the device for massaging the head, chest, genital area, or areas of skin with open wounds or damage. Do not use the device on post-surgical areas until tissues are fully healed.
7. Disassembly or self-repair of the device is prohibited. In case of malfunction, contact a MEDICA+ service center.

8. The device is equipped with a built-in rechargeable lithium battery.
9. Do not dispose of the device with household waste. At the end of its service life, the product must be delivered to a specialized collection and recycling facility for electrical and electronic equipment in accordance with applicable local regulations and environmental legislation.
10. Proper disposal helps prevent negative environmental and health impacts.

Device Description

The MEDICA+ MassHand Pro 2X percussion massager is a modern, high-performance device designed for deep muscle massage. It is developed for athletes, physically active individuals, massage therapists, and anyone who values high-quality muscle recovery.


Thanks to its powerful motor with a maximum power of 60W and operating speed from 1000 to 3200 RPM, the device effectively relieves muscle tension, stiffness, and fatigue, improves blood circulation, and promotes faster recovery after workouts or physical exertion.

Percussion technology deeply affects muscle tissues, helping to reduce pain, activate trigger points, and improve muscle elasticity.

Who is MEDICA+ MassHand Pro 2X suitable for:

- Athletes and individuals regularly engaged in fitness, running, strength training, or other physical activities
- Accelerating muscle recovery after intense workouts
- People with sedentary jobs experiencing tension in the neck, shoulders, back, or lower back
- Those suffering from muscle stiffness and fatigue after physical exertion
- Massage therapists as an auxiliary tool for deep muscle treatment
- Individuals with active lifestyles requiring regular muscle relaxation at home
- Use in sports clubs, fitness studios, and rehabilitation centers (in the absence of contraindications)

Consult a physician before use if injuries or chronic conditions are present.

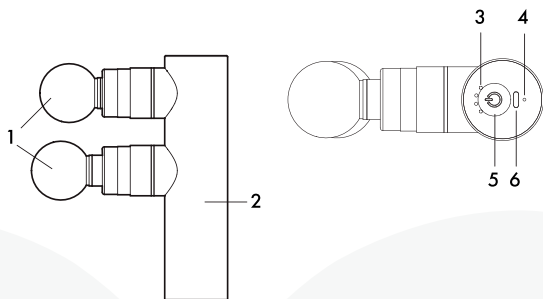1. Interchangeable massage attachment
2. Device handle
3. Speed level indicator
4. Battery level indicator
5. Power/speed control button
6. USB charging port

Attachments

"Round" attachment – 2 pcs.

Designed for soft and gentle massage of sensitive or painful body areas. Ideal for large muscle groups such as thighs, glutes, arms, and calves, ensuring even pressure distribution without excessive force.

"Fork" attachment

Specially designed for massage along the spine, neck, and Achilles tendon. Its shape allows gentle treatment of muscles on both sides of the spinal column while avoiding direct pressure on bone structures. Recommended for relieving tension in the neck and lower back.

“Bullet” attachment

Provides deep and concentrated impact on specific muscle points. Designed for trigger points, tension knots, and small muscle groups, including hands, feet, shoulders, and forearms. Recommended for deep localized massage.

“Flat” attachment

It is versatile and suitable for general massage of large muscle groups such as the back, chest muscles, buttocks, and thighs. It provides a moderate level of intensity and even contact with the body surface, making it an optimal choice for daily use.

 **Warning:** Do not change attachments while the device is operating.

Charging

Fully charge the device before first use.

1. Connect the included USB cable to a power adapter (adapter not included) and to the port on the massager.
2. Battery indicator status:
 - Flashing red light — low battery
 - Solid red light — charging
 - Solid green light — fully charged
3. Full charging time: approximately 3 hours. After full charge, the device can operate up to 4 hours without connection to power.

Note: Do not charge in direct sunlight or near heating devices. Use only the recommended 5V/2A adapter. Do not use fast chargers (9V or 12V) to avoid battery damage.

Low Battery Protection

- When battery voltage drops to 7.3V, the indicator flashes red.
- Below 7V, the device automatically shuts off.

Maximum Operation Time Protection

- The device automatically turns off after 10 minutes of continuous operation.
- To restart, press and hold the power button for 2 seconds and select the desired speed level.

Instructions for Use

1. Insert the selected attachment into the device until fully secured.
2. Press and hold the power/speed button for 2 seconds to turn on the massager.
3. Initial speed level is set to 1.
4. Each subsequent press changes speed in ascending order: 1 → 2 → 3 → 4 → 5.

Speed 1 — 2000 RPM (33 Hz): warm-up or massage of very sensitive areas.

Speed 2 — 2500 RPM (42 Hz): relief of fatigue and moderate muscle pain.

Speed 3 — 3000 RPM (50 Hz): relaxation of tense and stiff muscles.

Speed 4 — 3200 RPM (53 Hz): deep massage of dense knots and trigger points.

Speed 5 — 2000–3200 RPM (33–53 Hz): automatic adjustment depending on applied pressure.

Note: The higher the speed, the more intensive the massage.

5. Before massage, it is recommended to apply massage cream to the selected area. The cream ensures optimal glide. The massage cream can be used with all attachments except the round attachment, as it absorbs cream and may lose effectiveness.
6. Apply the attachment to soft tissues.
7. Perform massage with smooth movements, avoiding prolonged pressure on one spot.
8. Recommended treatment time per area: 1–2 minutes.

Contraindications

Before using the MEDICA+ MassHand Pro 2X percussion massager, it is recommended that you familiarize yourself with the following restrictions. Use of the device by the following persons may lead to deterioration of health or injury:


- Pregnancy and lactation – massage can affect blood circulation and muscle tone, which is undesirable for the mother and child.
- Diabetes mellitus, sensory disorders, or peripheral neuropathies – there is a risk of skin and muscle injury due to reduced sensory response.
- Presence of internal medical devices (pacemaker, defibrillator).
- Acute injuries, inflammatory processes, burns, or skin infections.
- Tumors or oncological diseases.
- Chronic cardiovascular diseases or high blood pressure – prior consultation with a doctor is necessary.
- Elevated temperature, fever, or general malaise.
- Use on the head, chest, and genitals.
- Children under 14 years of age without adult supervision.

Note: Before using the massager on any part of the body in the presence of chronic or serious diseases, as well as if you have doubts about your health, you should consult your doctor.

Heat/Cold Attachment

This attachment is not included in the standard MEDICA+ MassHand Pro 2X massager set, but it can be purchased separately. It is fully compatible with this model and can be used with all standard massager attachments.

The attachment provides heating and cooling functions, allowing you to combine massage with thermal procedures to increase muscle relaxation and improve blood circulation. Before use, make sure that the attachment is securely fastened.

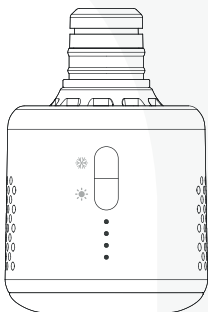Warranty Terms

Dear Customers!

Please keep these instructions in a safe place so that you can use the warranty card. The warranty is valid for one year from the date of purchase (or for 2 or 3 years if you purchase an extended warranty) in case of failure under normal operating conditions.

The warranty does not cover the following cases:

1. If the product is damaged due to misuse or lack of proper maintenance.
2. If the product is damaged as a result of disassembly or modification.
3. If parts that do not belong to our company have been applied and used in the device.
4. Damage caused by force majeure.

According to the terms and conditions, maintenance provided outside the warranty is subject to additional payment.

Parts of the product replaced during the warranty period remain the property of MEDICA+.

Technical specifications

Dimensions: 175 × 45 × 92 mm

Weight: 536 g

Number of speed modes: 5

Speed range: Speed 1 — 2000 rpm (33 Hz), speed 2 — 2500 rpm (42 Hz), speed 3 — 3000 rpm (50 Hz), speed 4 — 3200 rpm (53 Hz), speed 5 — 2000–3200 rpm (33–53 Hz)

Maximum continuous operation time per session: up to 10 minutes

Power supply type: rechargeable battery

Charging time: approximately 3 hours

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