ENG / UA/ PL

**Heating and Cooling Attachment
for MassHand Pro 2X**

medica+ 
JAPAN TECHNOLOGY

ENG

Instruction

**Heating and Cooling
Attachment
for MassHand Pro 2X**

Content

Safety Instructions	01-02
Description	02-03
Heating Button.....	03
Cooling Button.....	04
LED Indicators	04
Warranty Terms.....	05
Technical Specifications	06

Dear Customer,
Thank you for choosing MEDICA+ products.
Please read this instruction manual carefully before using the device.

Safety Instructions

1. The attachment is intended exclusively for thermal heating and cooling massage of muscle tissues and fascia. Do not use the attachment for any other purposes. Improper use may result in device damage or personal injury.
2. Do not change the attachment while the massager is in operation.
3. Do not wash or immerse the attachment in water or any other liquids. The attachment contains electronic components and a built-in battery; contact with liquids may cause a short circuit, electric shock, or permanent damage to the device. For cleaning, use a dry or slightly damp cloth, avoiding direct contact of water with electronic components.
4. Keep the device and attachment out of reach of children to prevent accidental activation, burns, or injury.


5. Always monitor a comfortable temperature and massage intensity. Do not apply heat or cooling to the same area of skin for an extended period. Please note that the attachment has maximum operating time limits: up to 5 minutes for cooling and up to 10 minutes for heating, after which automatic shut-off will activate.

Description

The Heating and Cooling Attachment for MEDICA+ MassHand Pro 2X (sold separately) is an additional accessory that expands the functional capabilities of the device. It is fully compatible with this model and can be easily installed in place of the standard attachments.

The attachment is equipped with both heating and cooling functions, allowing you to combine percussion massage with thermal therapy. The heating mode promotes deeper muscle relaxation and reduces the sensation of tension, while the cooling mode helps tone tissues after physical activity.

Suitable for use on various areas of the body in accordance with the massager operating guidelines.

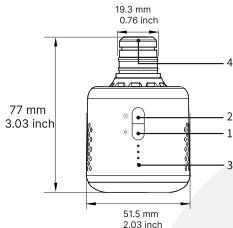1. Heating Button
2. Cooling Button
3. Indicator Lights
4. USB Type-C Charging Port

Heating Button

1. Turning on heating: Press and hold the button for 1.5 seconds.
2. Adjusting heating levels: Short presses switch between four intensity levels.
3. Turning off heating: Press and hold the button for 1.5 seconds.

Cooling Button

1. Turning on cooling: Press and hold the button for 1.5 seconds.
2. Adjusting cooling levels: Short presses switch between four intensity levels.
3. Turning off cooling: Press and hold any button (heating/cooling) for 1.5 seconds.

LED Indicators

- Charging: Four red LEDs flash during charging. A steady blue light indicates a full charge.
- Cooling levels: Each level corresponds to a specific blue LED.
- Heating levels: Each level corresponds to a specific red LED.
- Low battery: The first LED flashes red.

Warranty Terms

Dear Customers,

Please keep this instruction manual properly in order to use the warranty certificate. The warranty is valid for one year from the date of purchase (or for 2 or 3 years if an extended warranty has been purchased) in the event of a malfunction under normal operating conditions.

The warranty does not cover the following cases:

1. If the product is damaged due to improper use or lack of proper maintenance.
2. If the product is damaged as a result of disassembly or modification.
3. If parts not supplied or approved by our company have been used with the device.
4. Damage caused by force majeure circumstances.

According to these terms, maintenance services provided outside the warranty period are subject to additional charges.

Parts replaced during the warranty period remain the property of MEDICA+.

Technical Specifications

Display: LED

Cooling Temperature: 8–14°C (4 levels)

Heating Temperature: 39–45°C (4 levels)

Temperature Accuracy: $\pm 3^{\circ}\text{C}$

Battery: USB Type-C 5V=1A, 700 mAh capacity

Operating Time: up to 30 minutes (cooling) or 100 minutes (heating)

Auto Shut-Off: 5 minutes (cooling), 10 minutes (heating)

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