ENG / UA / PL

AIR Massage 10X

medica+ 
JAPAN TECHNOLOGY

ENG

Instruction manual

**MEDICA+ AIR Massage 10X
Lymphatic Drainage Device**

Content

Safety precautions for using the massager.....	01-02
Device description.....	02-04
Warnings and contraindications.....	05-06
Main unit.....	07
Battery charge indicator.....	08
Using the device	09-11
Massage modes	12-15
Care and cleaning.....	16
Warranty terms.....	17
Technical specifications	18

Dear Customer,

Thank you for choosing MEDICA+ products.

Please read this manual carefully before using the device.

Safety Precautions for Using the Massager

1. Do not use power adapters that do not meet the technical requirements of this device.
2. Do not disassemble or attempt to repair the device yourself.
3. If the power cable is damaged, contact an authorized MEDICA+ service center.
4. Do not use the device in a humid environment or if water or other liquids have come into contact with it.
5. After use, store the massager in a dry place, away from moisture and direct sunlight.
6. Do not use or store the device near flammable, explosive substances, or chemical solutions.
7. Avoid contact of the device with sharp objects to prevent damage to

- its components.
8. If you experience discomfort or unusual sensations during use, stop the procedure immediately.
 9. Do not use the device at high temperatures or in overheated environments.
 10. Keep the device clean; use a dry, soft cloth for care and maintenance.

Device Description

The MEDICA+ AIR Massage 10X Lymphatic Drainage Device is a modern intelligent device designed for effective muscle relaxation and recovery of the legs after physical exertion. It operates using air compression technology that replicates the movements of professional manual massage, providing a deep yet comfortable effect. The device combines advanced compression therapy technologies with the convenience of home use.

Thanks to its multi-chamber system, the device evenly covers the entire leg—from the feet and calves to the knee and thigh

areas. During operation, the chambers sequentially inflate and deflate, creating gentle rhythmic pressure. This helps stimulate blood circulation, reduce fatigue, and effectively relieve muscle tension and heaviness in the legs.

1

2

3

4

5

6

1. Main unit — 1 pc.
2. Charging cable — 1 pc.
3. Connecting hose — 1 pc.
4. Massage cuffs (boots) — 2 pcs.
5. User manual — 1 pc.
6. Packaging (box) — 1 pc.

The manufacturer reserves the right to modify the package contents and design elements of the device in order to improve its performance and quality without prior notice to the user.

Warnings and Contraindications

It is recommended to consult a doctor or use the device under the supervision of a specialist before use.

Do not use the device in the following cases:

- deep vein thrombosis or suspected thrombosis;
- thrombophlebitis;
- severe cardiovascular diseases;
- pronounced swelling associated with heart or kidney failure;
- acute inflammatory processes in the body;
- oncological diseases (without a doctor's approval);
- skin damage in the treatment area (open wounds, burns, infections);
- complicated varicose veins (inflammation, pain, presence of blood clots);
- impaired sensitivity or numbness in the legs.

Use Only After Consulting a Doctor:


- pregnant women;
- individuals who have recently undergone surgery;
- people with pacemakers;
- users with implanted metal devices (prostheses, etc.);
- individuals with unstable blood pressure;
- people with diabetes (especially with complications);
- individuals with chronic diseases;
- people with increased sensitivity to pressure or discomfort during massage.

Additionally:

- children may use the device only under adult supervision;
- if pain, discomfort, or unusual sensations occur during the procedure, stop using the device immediately.

Main Unit of the Device

The device is equipped with an informative display that shows key operating parameters, including pressure level, mode, time, and battery status.

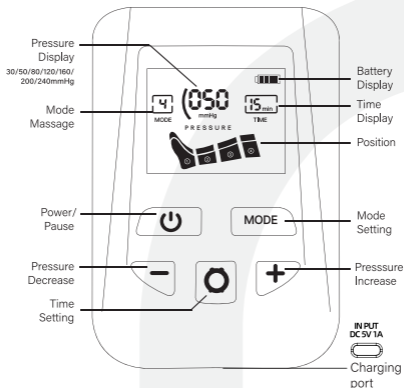Thanks to the intuitive control panel and clear display, you can easily select personalized settings for the most comfortable and effective massage.

Battery Charge Indicator

The battery level is shown using a color indicator:

- Green indicator: 100% – 65% charge
- Blue indicator: 65% – 25% charge
- Red indicator: 25% – 5% charge

If the battery level drops below 5%, the device automatically turns off to protect the battery.

During charging, the display shows “CH”. Please note: the device must not be used while charging.

Using the Device

Preparation for Use

1. Before first use, fully charge the device (approximately 2 hours). Please note that the device cannot be used while charging.
2. Check the battery level using the indicator color (Green: 100–65%, Blue: 65–25%, Red: 25–5%).
3. Put on the cuffs/boots on your legs and connect them to the main unit using the connecting hose included in the package.
4. Turn on the device by pressing the power button  on the main unit display.

Operating Instructions

Step 1: Adjust the pressure.

Use the “+” or “-” buttons on the main unit display to select the intensity (from 30 to 240 mmHg).

Tip for beginners!

For the first use, it is recommended to start with a low pressure (around 80

mmHg) and then gradually adjust it to a comfortable level. Use higher levels only after you have adapted.

Step 2: Set the timer.

Press the time setting button  to select the session duration: 15, 30, or 45 minutes.




Step 3: Select a mode.

Press the “MODE” button to switch between 5 massage modes (from automatic cyclic mode to targeted zone massage).

The device will begin to sequentially inflate and deflate the chambers, creating moderate pressure to improve blood circulation and relieve fatigue.

Step 4:

After use, carefully disconnect the connecting hose from the main unit. Make sure to press the clips on both sides of the connector to avoid damaging it.

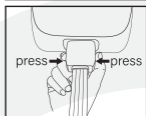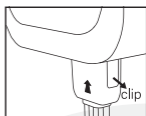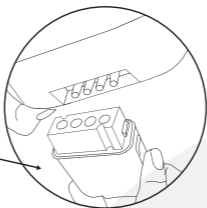Massage Modes

The device features 5 different compression algorithms that clearly demonstrate how the air chambers inflate and deflate during a session.

Mode 1 – Wave Massage

The chambers inflate strictly in sequence—one at a time from bottom to top. First, the foot is compressed, then it deflates, followed by the calf, then the knee, and finally the thigh.

Purpose:

- relieving mild fatigue at the end of the day;
- preparing muscles for a more intensive massage;
- suitable for people with high sensitivity to pressure.

Mode 2 – Sequential Compression

The chambers gradually inflate from bottom to top but do not deflate until the entire leg is fully compressed (foot → foot + calf → foot + calf + knee → entire leg).

Purpose:

- effective lymphatic drainage;
- reducing swelling after prolonged sitting or standing;
- improving venous return (prevention of blood stagnation).

Mode 3 – Double Wave

Two adjacent chambers inflate simultaneously, and this “pair” moves upward (foot + calf → calf + knee → knee + thigh).

Purpose:

- deep muscle treatment;
- relief of muscle spasms and the feeling of “heavy legs”;
- relaxation after prolonged walking.

Mode 4 – Full Compression

All four chambers (from foot to thigh) inflate simultaneously, hold the pressure, and then deflate at the same time.

Purpose:

- rapid vascular toning;
- deep relaxation;
- suitable for users who prefer strong pressure across the entire leg.

Mode 5 – Combined

An intelligent combination of the “Sequential Compression” and “Double

Wave" modes within a single session.

Purpose:

- maximum recovery after intense physical activity or sports;
- professional-level relaxation at home;
- achieving the best therapeutic effect in a single session.

Care and Cleaning

Main unit: wipe the display with a soft, dry cloth. Do not use abrasive materials or aggressive cleaning agents.

Fabric components (cuffs/boots): these parts are not machine washable. For cleaning, you may use alcohol wipes or clean, slightly damp wipes. After cleaning, allow all components to dry completely before further use.

Storage Conditions

Store the device in a dry, clean place out of reach of children. Avoid exposure to moisture, high temperatures, direct sunlight, and contact with water.

If the device is not used for an extended period, it is recommended to recharge it every 2 months to preserve battery life.

Disposal

Used batteries and electronic components must be disposed of in accordance with local environmental regulations and electronic waste recycling rules. Do not dispose of the device with household waste.

Warranty Terms

Dear Customer,
Please keep this manual in a safe place in order to use the warranty certificate. The warranty is provided for a period of one year from the date of purchase (or for 2 or 3 years if an extended warranty has been purchased) in case of failure under normal operating conditions.

The warranty does not apply in the following cases:

- If the product has been damaged due to improper use or lack of proper maintenance.
- If the product has been damaged as a result of disassembly or modification.
- If parts not supplied or approved by our company have been used with the device.
- Damage caused by force majeure circumstances.

According to the terms, out-of-warranty service is subject to additional charges. Parts replaced during the warranty period are not returned to the user.

Technical Specifications

Material: ABS + nylon

Pressure levels: 30 / 50 / 80 / 120 / 160 / 200 / 240 mmHg

Noise level: ≤ 55 dB

Charging port type: DC

Battery: lithium, 3200 mAh / 7.4 V

Input voltage: 100–240 V (AC)

Power: 24 W

Charging current: 5 V / 2 A

Charging time: approx. 2 hours

Operating time: 1.5–2 hours

Device weight: 1.4 kg

Maximum leg circumference: 70 cm

Minimum leg circumference: 25 cm

Operating conditions (use):

- Temperature range: from 5°C to 40°C
- Humidity range: 5% ~ 93% RH
- Atmospheric pressure: 86 kPa ~ 105 kPa

Storage and transportation conditions:

- Temperature range: from -10°C to +60°C
- Humidity range: 5% ~ 90% RH
- Atmospheric pressure: 86 kPa ~ 105 kPa

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